


Quick Guide - Get started with Bluetooth Solution Kit

1. Open the Jabra BT3030 box. Take off the chain/lanyard (A) by pulling it apart - it is not needed.
2. Charge the BT3030 Bluetooth device if it is not fully loaded.
3. Attach the BT3030 Bluetooth device to the neck loop with the aid of the clip (B) that is enclosed in the Jabra BT3030 box. See picture (C) and (D). *
4. Connect the neck loop's cable to the BT3030 Bluetooth device. (E)
5. Set your hearing instrument in T mode.
6. For instructions on handling of the BT3030 Bluetooth device, see the accompanying manual "Quick Start Guide" or the "User Manual" on the CD. It is also available at www.gnresound-ald.com, see Bluetooth Solution Kit.

**Alternative ways of wearing Bluetooth Solution Kit is to wear the neck loop around the neck as usual, but clip on the BT3030 Bluetooth device to your shirt's front pocket, border of the shirt or the like.*

NB! The BT3030 device has to be fairly close to your mouth since it contains a microphone that transmits your speech.

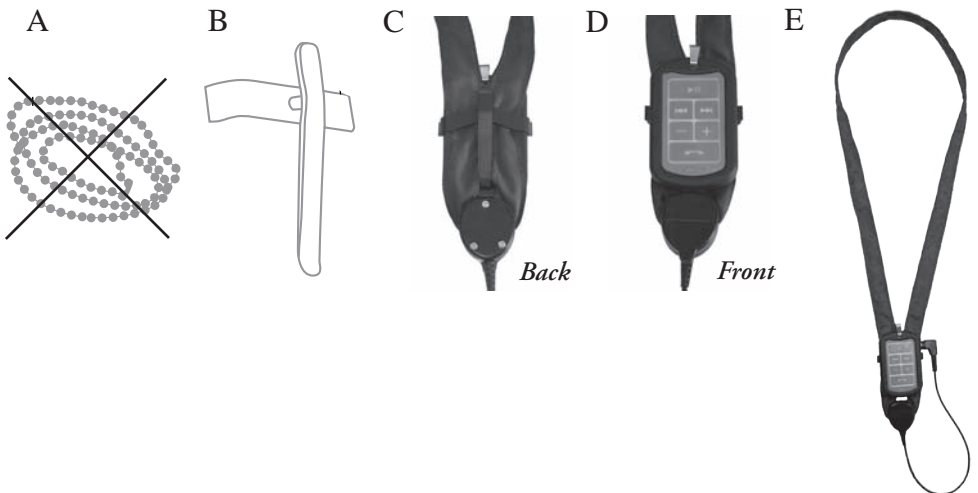

*Bluetooth Solution Kit
- ready to wear and use!*