CONFERENCE™

ENGLISH

A conversation amplifier with two microphones and an inductive loop built into an elegant folder.

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
Sidan 11

Ordering number: 415 04 04

Conversation amplifier with built-in inductive loop

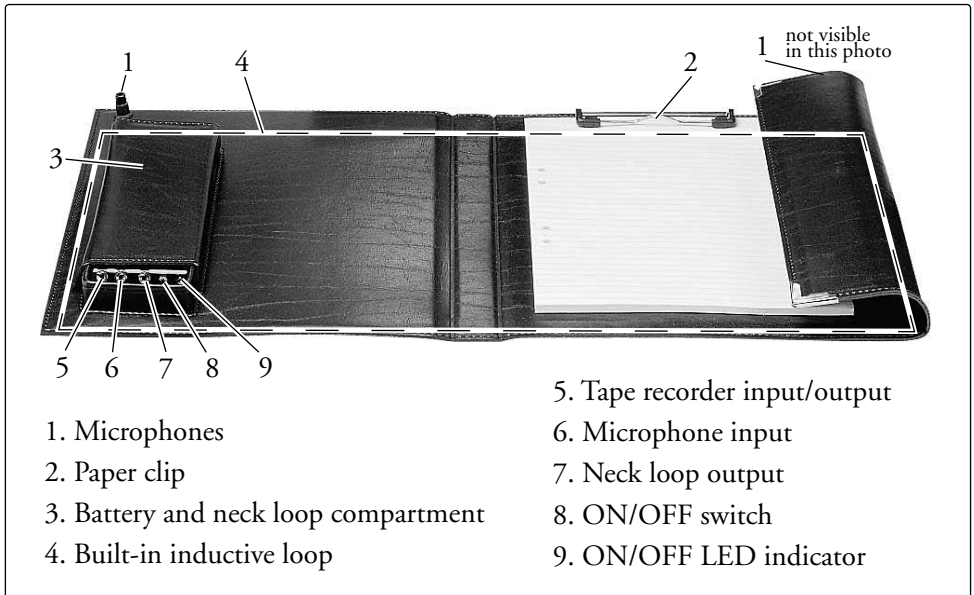The Conference™ is a conversation amplifier with two microphones and an inductive loop built into an elegant conference folder. It is very discreet and can be used at meetings and conversations at home and at work.

1. Microphones

2. Paper clip

3. Battery and neck loop compartment

4. Built-in inductive loop

5. Tape recorder input/output

6. Microphone input

7. Neck loop output

8. ON/OFF switch

9. ON/OFF LED indicator

Using the Conference

Put the Conference on the table and unfold it. Put the ON/OFF switch to the ON position. The red LED goes on. Switch the hearing instrument to T-position or MT-position.

The inductive loop is built into the Conference. Your position should be right in front of the folder to get the best transmission to the hearing instrument. Make sure the Conference is turned off when not in use (to save batteries).

Neck loop

In the compartment to the left, you will find a neck loop to the Conference. The neck loop gives more freedom of movement because the inductive loop is closer to the hearing aid and has a cord to the Conference. Connect it to the T output. The internal inductive loop disconnects automatically.

External microphone

If you want shorter distance to the person speaking, you can use an external microphone instead of the built-in microphones. Connect it to the MIC input. The internal microphones disconnect automatically.

Recording a meeting

It is possible to record the conversation from a meeting. The Conference is equipped with a 3.5 mm jack for both recording and playback in mono. The cord to the recording device should be wired according to technical data on the last page.

Changing batteries

Open the compartment to the left and replace the old batteries with new ones (type LR6/AA).

Troubleshooting

Type of fault	Cause	Action
The red diode does not light up when Conference is switched on.	<ol style="list-style-type: none"> 1. No batteries have been inserted. 2. The batteries are not fitted correctly. 3. The batteries are flat. 	<ol style="list-style-type: none"> 1. Insert the batteries. 2. Press the batteries properly into the battery holder. 3. Change the batteries.
The sound from the built-in loop is difficult to hear.	<ol style="list-style-type: none"> 1. The hearing instrument is not in the T position. 2. You are sitting too far away from Conference. 	<ol style="list-style-type: none"> 1. Switch to the T or MT position. 2. Move your head nearer to Conference until you can hear the sound well again or use the neck loop supplied.
The sound from the neck loop is weak.	The volume control on the neck loop has been turned down.	Adjust the volume to a comfortable level.

Cleaning and maintenance

Use a damp cloth to wipe the product on the outside. Never use cleaning products or solvents. From time to time check that the cables and connections are intact. Refer to the technical data for the battery service life. Refer to page 3 for how to change the battery.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

Low-energy radiation (eg from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.

Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Technical data

Frequency range:	200-8000 Hz, -3 dB rel 1 kHz.
Microphones:	2 pc built-in.
Connectors:	External microphone: in. Neck loop: out. Tape recorder: in/out (3.5 mm, input: tip, output: ring).
Power supply:	4 pcs LR6/AA 1.5 V penlight.
Operating time:	Approx. 100 hours.
Field strength:	+12 dB rel 100 mA/m at 0.7 mV (70 dB SPL) from the microphones at a distance of 45 cm from the loop.
Distortion:	<3%.
Gain:	0.7 mV from the microphones produces 250 mV out at the tape recording. The tape contact can also be used for playback.
Current consumption:	25-200 mA.
Weight:	Approx. 1.3 kg.

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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GN ReSound AB
GN ReSound AB ALD Division
INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**