 **Lynx™** Door Transmitter


Ordering number: 200 20 94

<b>ENGLISH: Door transmitter for the Lynx system .....</b>	<b>page 3</b>
<b>DEUTSCH: Türmelder für das Lynx System .....</b>	<b>Seite 8</b>
<b>FRANÇAIS: Émetteur de porte d'entrée pour le système Lynx.....</b>	<b>page 13</b>
<b>ITALIANO: Trasmettitore porta per il sistema Lynx .....</b>	<b>Pag. 18</b>
<b>NEDERLANDS: Deurbelzender voor het Lynx-systeem .....</b>	<b>Pag. 23</b>
<b>NORSK: Dørsender til Lynx-systemet .....</b>	<b>side 28</b>
<b>SVENSKA: Dörrsändare till vårt varseblivningssystem Lynx .....</b>	<b>sid 33</b>

## Door transmitter for the Lynx system

The door transmitter forms part of Lynx – our wireless alert system for people with impaired hearing. Lynx uses various types of transmitters and receivers that alert you to noises such as a ringing telephone or doorbell, or the first signs of a fire.

The door transmitter is designed like a standard doorbell. When the button is pressed, the signal is sent wirelessly to the receiver in the system. One or more repeaters may also be installed in order to amplify the alarm signal and thus extend its range.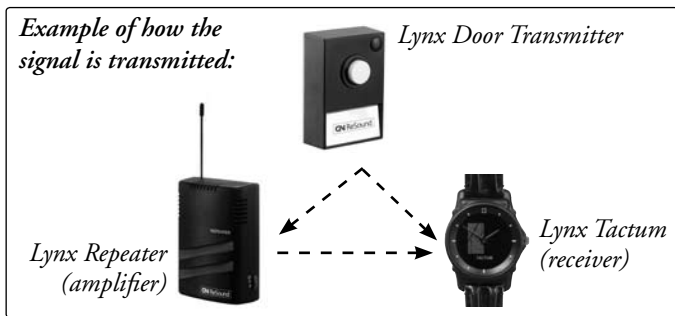


### Connection options

It is simplest to place the door transmitter alongside the normal doorbell (see next page). The normal doorbell will emit an audible signal as before and the Lynx Door Transmitter will send out an alert to the selected receiver in the Lynx system. If you prefer to replace the old doorbell with the Lynx Door Transmitter, there are two options:

- 1) Connect the Door Transmitter to the existing doorbell (see next page).
- 2) Install a Lynx Repeater (amplifier) which will emit an audible signal when someone rings the doorbell (see Repeater user manual).

## Connecting alongside the existing doorbell

The door transmitter can either be fastened using double-sided adhesive tape or screwed on. Screw on the door transmitter from the front using the through screws. Remove the plug for the upper screw hole. **NOTE:** two screws on the rear must be removed before the door transmitter can be screwed on.
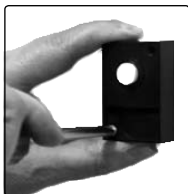


## Connecting to the existing doorbell


The existing doorbell will ring and the receiver in the Lynx system will send out an alert.

1. Unscrew the door transmitter.
2. Connect the two cables which are attached in the existing doorbell to the connecting strip in the door transmitter.

*NOTE: This connection requires a transformer which complies with standard EN 60558-1/60558-2-6 and is fused on the door transmitter side with a 2A fuse (see figure 2a).*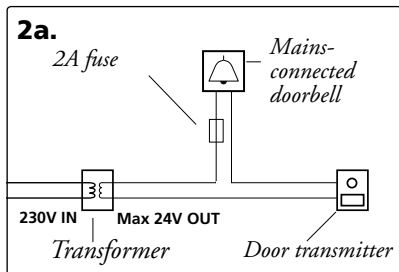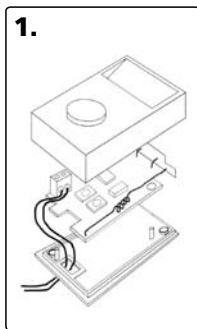

Connection to mains-connected doorbell

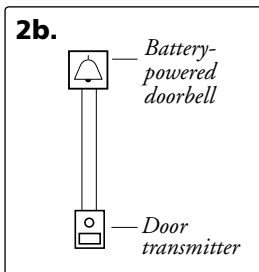Connection to battery-powered doorbell

## When the doorbell rings

When someone rings the doorbell, the receiver installed in the Lynx system will be activated and will issue a warning by vibration and symbols, loud noise, or flashing light, depending on the type of receiver.

**Read the user manual for the receiver installed!**

## Battery warning

The door transmitter is powered by a battery with a normal service life of approximately 5 years. When battery power begins to fail, the door transmitter will send out an alert. It sends a flat-battery warning along with the alarm to the receiver in the alerting system.


**Read the user manual for the receiver installed!**

## Changing the battery

1. Unscrew the two screws on the rear of the door transmitter, and release the rear cover by lifting it straight up.
2. Carefully push the old battery out of the holder.
3. Replace the battery with a new CR2032 Duracell type battery.

**Note: The battery's positive terminal must be facing upwards.**

4. Screw on the rear cover.

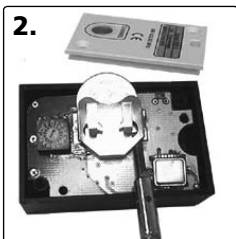## Changing the channel setting

The door transmitter sends signals wirelessly to the receiver installed in the Lynx system. It can transmit on a total of 16 different channels (0-9, A-F). Channel 0 is reserved for a common fire alarm and should not be used. Transmitters and receivers in the system must be set to the same channel so that they can communicate. They are all pre-set to channel 9, and then operate together without alteration.

In some cases, it may be necessary to change the channel setting. If two or more users live near one another, the alarms can interfere. This can be rectified by changing the channel settings so that they are different for everyone. Any channel can be selected except Channel 0.

If you need to change channels on the receiver, see the receiver user manual. If you need to change the channel on the door transmitter, proceed as follows:

The channel selector is on the rear. Carefully release the plastic protection over it.

Use a small screwdriver and turn the yellow arrow towards the required channel. Refit the plastic protection over the channel selector.

## Technical data

Modulation:	AM.
Frequency:	433.92 MHz.
Power supply:	3 V lithium battery. <b>WARNING:</b> Risk of explosion if fitted incorrectly!
Service life:	Approx 5 years (with 3 alarms per day). Maximum 70,000 alarms.
Range:	Up to approx. 30 m indoors.
Channels:	16.
Dimensions:	60 x 40 x 23 mm.
Weight:	35 g.

**CE** Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

## Troubleshooting

Type of fault	Action
The door transmitter gives a false alarm.	Someone in the vicinity has the same system installed. Change to another channel on all units.
Receiver fails to respond.	<ol style="list-style-type: none"> <li>1. Check the range. Try to reduce the distance between the door transmitter and the receiver or install a Repeater to extend the range.</li> <li>2. Check that the receiver's battery is OK.</li> </ol>

## Cleaning and maintenance

Use a damp cloth to wipe it on the outside. Never use cleaning products or solvents.

The battery will last approx. 5 years. Refer to page 5 for how to change the battery.

## Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

## Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

## General precautions

- Low-energy radiation (eg from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

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**GN ReSound AB